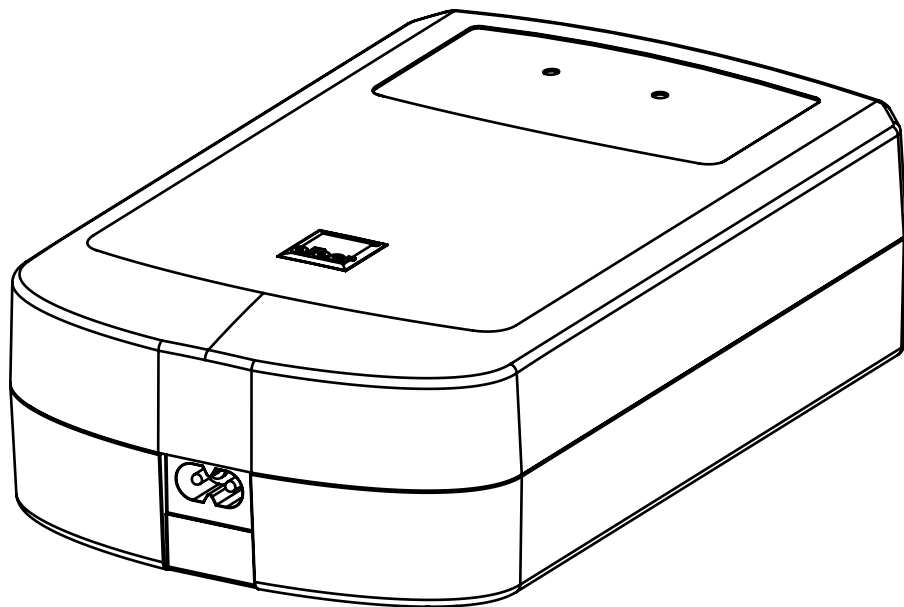
alber

PS4820

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SAFETY INSTRUCTIONS

- Please read these operating instructions before using the device.
- The PS4820 charger is only permitted for charging the e-motion M25.
- Use with other products is not permitted and can result in significant damage to this charger and to the other equipment.
- Use the charger only if the connecting cables and the housing are undamaged.
- Do not pull on the connecting cables as this could damage or break the wires.
- The device is intended for use in dry rooms and not permitted to be used outdoors.
- If the charger is moved from a cold environment to a warm room, condensate may form. Only use the charger once it is completely dry inside and out! This could take several hours.
- Ensure there is adequate ventilation when using the device.
- Do not move the device along with the wheelchair.
- When charging is complete, remove the mains plug from the mains outlet and disconnect the charge cables from the e-motion drive wheel.
- When using the device, set it down on its underside and ensure that the device is not covered.
- Position the device such that the mains plug can be unplugged from the mains outlet at any time as this is used to switch the device off.
- Do not open the device.
- Supervise children and keep them away from the device when it is being used.
- Volatile gases may be produced during charging. Do not use the device in the vicinity of naked flames or ignition sources.
- Do not make any unauthorised changes to the device!

PURPOSE

The PS4820 charger is intended for charging the drive wheels of the e-motion M25. The charger is designed for stationary use and is not suitable for being moved along with the wheelchair. The charger can be used throughout the world and automatically adapts to the respective mains voltage. Please use a corresponding mains adapter if required. The proper charge procedure is primarily determined by the rechargeable battery itself. For this reason, please pay attention to the charge monitoring display on the e-motion drive wheels. If necessary, please check the electrical connections and ensure that the charger is used free standing.

OPERATING MODE DISPLAY

LED Colour	LED Mode	Status
Red	Illuminated	Fault
Orange	Illuminated	Ready for use
Green	Flashing	Charging in progress
Green	Illuminated	Charging completed

FIRST USE

1. Plug the mains plug into the mains outlet. Both status indicators on the charger illuminate orange.
2. Connect the charger with the two drive wheels. The magnetic contact of the two charge plugs ensures an 'automatic' connection with the charge sockets of the e-motion drive wheels. Both status indicators on the charger flash green.
3. Observe the charge status indication on the charger and on the e-motion drive wheel. The drive wheel shall not be moved during the charge process.
4. Unplug the mains plug from the mains outlet once both charge status indicators on the charger indicate "full" with a constant green illumination. In urgent cases the charge process can be interrupted at any time.
5. Disconnect the charge cable from the rechargeable battery pack.

MAINTENANCE AND CARE

The charger is maintenance-free. If necessary, clean the device only with a damp cloth. Do not use aggressive cleaning agents. It is not permitted for any liquid to enter the interior of the charger. Protect the mains plug and charge connectors from contamination.

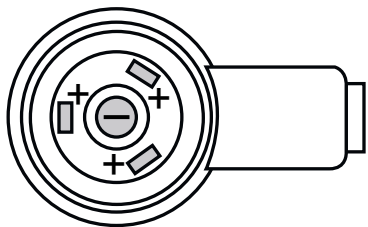
GUARANTEE

We provide a two-year warranty for the product from the date of purchase.

TECHNICAL DATA

Model: PS4820
 Rated Input Voltage: 100 - 240VAC, 50Hz - 60Hz
 Input Current: Max. 2,5A
 Rated Output Voltage: 2 x 48VDC
 Rated Output Current: 2 x 1,0 A
 Rated Output Power: 96 W
 Protection Type: IP 31
 Protection Class: II
 Operating Temperature: 0 - 45°C
 Humidity: Storage 5% - 95%
 Operation 10% - 80%
 Air Pressure: Storage 500 - 1060 hPa
 Operation 700 - 1060 hPa
 Operating altitude: max. 2000m
 Storage Temperature: -40°C - +65°C

Polarity of the two charge plugs:

"TECHNICAL DATA" LABEL

The label shown on the next page can be found on the underside of the charger and contains its technical data. Means:

- Compliant with European directives
- Compliant with US and Canadian directives
- Protective insulation / device class 2
- Do not use outdoors
- Protect the device from moisture
- Warning: Magnetic fields and forces
- Before using the charger: Read the operating instructions and observe the accompanying leaflet
- Do not dispose of in household waste
- Temperature range for operation

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Please act in an environmentally-conscious manner and bring this product to your regional recycling centre at the end of its service life. Please familiarise yourself with the applicable legislation in your country regarding disposal, because the WEEE Directive does not apply in all European States. These components are also alternatively taken back by Alber or Alber dealers for proper, environmentally sound disposal.

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OEM Hersteller / OEM Manufacturer ANSMANN AG
Alber GmbH